

Training Information Sheet

EleGARD™ Disposable Cover Installation Process

Note: The disposable cover is intended for single patient use and should be replaced after every patient contact, or if there are any breaks in the material. The disposable cover does not eliminate the need to clean and disinfect the EleGARD™ Patient Positioning System after every use.

After cleaning and disinfecting the EleGARD System according to agency protocol, ensure the device is completely dry before installing the disposable cover for next use.

Ensure:

- the LUCAS® back plate is secured in place.
- the head of the EleGARD is in the lowest position.
- the headrest is in the lowest position.

Legend of terms used in this sheet



This image shows cover right side up and upside down. Please note that the inside of the cover is darker and shiny.

INSTALLATION



1. Orient the cover next to the EleGARD™ System aligning the clear plastic end of the cover with the control panel.



2. Slip the pre-cut slots over the LUCAS attachment points.



3. Pull the blue end of the cover over the front/bullnose portion.



4. Invert the clear plastic end to locate the two short adhesive strips. Working one side at a time, secure adhesive strip to the top of the control panel.



5. Stretch the elastic over the red headrest adjuster keeping the clear plastic above the lift handles.



6. Ensure the LUCAS attachment points are accessible through the pre-cut slots and the control panel is visible through the plastic.

The EleGARD is now ready for use and may be placed back inside the carrying case.

REMOVAL and FURTHER INFORMATION

1. To remove the disposable cover, start at the control panel end of the EleGARD™ System.
2. Carefully lift the elastic along the bottom edge of the clear plastic, slowly turning the cover inside out while removing to assist in containing contaminants inside the cover.
3. Work slowly toward the opposite end to avoid spreading any body fluids or contaminants.
4. Continue this process until the cover is completely removed from the EleGARD.
5. Follow agency protocol for cleaning and disinfecting.
6. Ensure the EleGARD is completely dry before installing a clean cover.

To view a video on cover installation, scan this QR Code.



For questions, please email Clinical@ElevatedCPR.com

The EleGARD™ Disposable Cover is manufactured by Advance Medical Design, Inc. and distributed by AdvancedCPR Solutions® LLC