

EleGARD

After every use, clean the EleGARD and let it dry before returning it to the carrying case.

Caution: Liquid

Do not allow water or cleaning fluids to enter the EleGARD. Use cleaning liquids on a dampened cleaning cloth. Avoid liquid pooling on the EleGARD.

Caution: Abrasive Cleaners

Abrasive powders, steel wool, industrial strength or solvent cleaners, and vinyl “conditioners” or “protectors” should be avoided. Undiluted bleach or lacquer can dull or damage the product. If in doubt about a detergent or cleaner, test a patch in an inconspicuous place before applying in a larger place.

1. Raise the EleGARD to the fully raised position to clean.
2. Manually pull the gray headrest handle to extend the slide as needed to flatten the pleats in a smooth surface as needed for cleaning.
3. Disinfect all surfaces with a soft cloth and disinfectant. NOTE: It is important to clean the EleGARD after disinfecting per the instructions below. Recommended disinfectants include:

Asepticare TB-II Birex SE

Bleach 1.5 (20% bleach) Bleach 1.9 (10% bleach) Bleach Rite

Broad Spectrum Quaternary Disinfectant Cleaner

Cavicide

Cavicide1

Clorox Healthcare Bleach Germicidal Wipes

Clorox Healthcare Bleach Germicidal Cleaner

Clorox Hydrogen Peroxide Cleaner Disinfectant Wipes

Clorox Hydrogen Peroxide Cleaner Disinfectant Spray

EZ Kill Wipes OPTIM 33TB Oxivir TB

PDI Sani Cloth AF3 Germicidal Disposable Wipes

PDI Sani-Cloth HB

PDI Sani-Cloth Plus

PDI Super Sani-Cloth

PDI Super Sani Cloth Bleach Germicidal Disposable Wipes

Precise QTB Spray SaniZide Plus

Virox AHP 5

Virex II 256

Wex-Cide 128

4. Clean all surfaces with a soft cloth and warm water with a mild cleaning agent to remove chemical residue. Recommended cleaners include:

Formula 409 All-Purpose spray cleaner

Fantasik spray cleaner

Hand dishwashing soap and water

5. After cleaning, rinse all surfaces with a soft cloth dampened with water and then dry with a clean, lint-free cloth.

Caution: Chemical Residue

Failure to remove chemical residue will cause discoloration of the coated fabric surface to occur earlier in the life cycle of the product. Always clean after disinfecting.

LUCAS 3.1

Cleaning routines Clean all surfaces and straps with a soft cloth and warm water with a mild cleaning agent or disinfectant agent, e.g. • 70% isopropyl alcohol solution • 45% isopropyl alcohol with added detergent • Quaternary ammonium compound • 10% bleach • Peracetic (peroxide) acid solutions Follow the handling instructions from the manufacturer of the disinfectant. Caution - liquid Do not immerse the LUCAS Chest Compression System in liquid. The device can be damaged if liquid enters the hood. Allow the device to dry before you pack it into the Carrying Case.

ResQPOD ITD

The ResQPOD ITD 10 is a single person device. The device can be cleaned of secretions (e.g mucous, spit) by shaking the device. DO NOT submerge the device in water or place in a dishwasher for cleaning.