

# Training Information Sheet

## EleGARD™ Disposable Cover Installation Process

*Note: The disposable cover is intended for single patient use and should be replaced after every patient contact, or if there are any breaks in the material. The disposable cover does not eliminate the need to clean and disinfect the EleGARD™ Patient Positioning System after every use.*

After cleaning and disinfecting the EleGARD System according to agency protocol, ensure the device is completely dry before installing the disposable cover for next use.

Ensure:

- the LUCAS® back plate is secured in place.1
- the head of the EleGARD is in the lowest position.
- the headrest is in the lowest position.

### Legend of terms used in this sheet



*This image shows cover right side up and upside down. Please note that the inside of the cover is darker and shiny.*

# INSTALLATION



1. Orient the cover next to the EleGARD™ System aligning the clear plastic end of the cover with the control panel of the EleGARD.



2. Loop the blue end of the cover over the front/bull nose portion of the EleGARD so that the elastic overlaps the device by approximately 1-2 inches.



3. Locate the two short adhesive strips. Working one side at a time, secure adhesive strip to the top of the control panel. *Note: Disregard long horizontal adhesive strip. It is not needed for installation of the disposable cover*



4. Stretch the elastic over the red headrest adjuster, but not past the handle at the base of the unit.



5. The sides of the cover should be placed on top of the LUCAS back plate to ensure the attachment points for LUCAS claw locks remain clearly visible.



**The EleGARD is now ready for use and may be placed back inside the carrying case.**

## REMOVAL and FURTHER INFORMATION

---

1. To remove the disposable cover, start at the control panel end of the EleGARD™ System.
2. Carefully lift the elastic along the bottom edge of the clear plastic, slowly turning the cover inside out while removing to assist in containing contaminants inside the cover.
3. Work slowly toward the opposite end to avoid spreading any body fluids or contaminants.
4. Continue this process until the cover is completely removed from the EleGARD.
5. Follow agency protocol for cleaning and disinfecting.
6. Ensure the EleGARD is completely dry before installing a clean cover..

**Don't forget to complete your user feedback sheet!**

**For questions, please email [Clinical@ElevatedCPR.com](mailto:Clinical@ElevatedCPR.com).**

*The EleGARD™ Disposable Cover is manufactured by Advance Medical Design, Inc. and distributed by AdvancedCPR Solutions® LLC*